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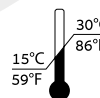
MicronJet 600™

Microneedles for Intradermal Injection

English

The MicronJet™

Microneedles for intradermal injection

For any further assistance/support/questions, please contact the manufacturer.

Manufacturer:

NanoPass Technologies Ltd.
3 Golda Meir St., Nes Ziona
7403648, Israel
Tel: +972-8-9462905
E-mail: info@nanopass.com

EU Representative:

MedNet EC-REP GmbH
Borkstrasse 10-48163, Muenster
Germany
Phone 49-251-322660
Fax 49-251-322662

MicronJet™ Instructions for Use.

Description:

- The MicronJet™ is a micro-needle device that is mounted on a standard syringe in the same manner as a conventional needle.
- The MicronJet™ comprises an integral array of 3 micro-needles; each micro-needle 0.6mm in length.
- The MicronJet™ is used to inject liquid substances, allowing for controlled intradermal delivery in any procedure which requires administration of substances to the dermal compartment.
- The MicronJet™ is latex free.

Indications:

The MicronJet™ is intended for intradermal injections of any substance or drug approved for delivery by this delivery route.

Contraindications:

- The MicronJet™ should not be used on skin abrasions, open wounds, cuts and scars.
- The MicronJet™ should not be used on rashes, skin infections or any other area of damaged or diseased skin.

Warnings and precautions:

- The MicronJet™ is intended for single use, by healthcare professionals only.
- The MicronJet™ can be used in all ages and populations.
- Re-use may lead to infection or other illness/injury.
- Used or un-used devices should not be re-capped or removed from syringe unless there is no alternative or such action is required by a specific medical procedure.
- The MicronJet™ is not intended for aspiration of liquids.
- If the device appears to be damaged or broken, discard and use a new device.
- Any serious incidence should be reported to the manufacturer and the competent Authority.

Adverse events:

Local adverse reactions, including edema, erythema and discoloration of skin at the injection site have been observed, most likely related to the injected substance, and not to the MicronJet™.

Adverse reactions related to use of the MicronJet™ include small risk of micro bleeding (self-limiting blood dotting, ceasing within normal clotting time). Other potential device related adverse events may rarely include local infection at the injection site.

Storage and handling:

- MicronJet™ should be stored at room temperature.
- The MicronJet™ is individually packed in a soft blister, sterilized by ethylene oxide (EtO).
- Do not use the MicronJet™, in the event that the soft blister is either open and/or damaged.
- Dispose of all devices in accordance with national/institutional guidelines, and in their absence, discard in a sharps container.

Supply method:

The device is supplied as a single sterile MicronJet™ device, packed in a soft blister for single use.

USA only - Caution: Federal (USA) law restricts this device to sale by or on the order of a licensed health care professional.

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